


PRONTO[®] FD kit

For the detection of the 2507+6 T>C mutation in the IKBKAP gene

REF 9948

96

Instructions for Use

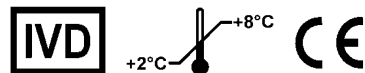

INTENDED USE

The PRONTO[®] FD kit is a Single Nucleotide Primer Extension ELISA Assay intended for the qualitative *in vitro* detection of the 2507+6 T>C mutation in the IKBKAP gene, from amplified human DNA.

For *in vitro* diagnostic use

BACKGROUND

Familial Dysautonomia (FD), also known as “Riley-Day Syndrome” or “Hereditary Sensory Neuropathy Type III”, is an autosomal recessive disorder that affects the development and survival of sensory, sympathetic, and some parasympathetic neurons. Individuals with FD are affected with a variety of symptoms, which include decreased sensitivity to pain and temperature, cardiovascular instability, recurrent pneumonias, vomiting crises, and gastrointestinal dysfunction^{1, 2, 3, 4}.

The major haplotype of FD is associated with mutation (2507+6T>C) that affects the donor splice site of intron 20 of the IKBKAP gene^{6, 7}.

FD disorder is primarily confined to individuals of Ashkenazi Jewish descent. The predicted carrier frequency of the defective gene is 1:30 within the Ashkenazi Jewish population⁵.

REFERENCES

1. Riley CM, Day RI, Greely D, Langford WS Central autonomic dysfunction with defective lacrimation. *Pediatrics* 3: 468-477 (1949).
2. Axelrod FC, Nachrigal R, Dancis J. FD diagnosis, pathogenesis and management. *Adv Podiatry* 21: 75-96 (1974).
3. Axelrod FB FD In: Robebertson D, Low PA, Polinsky RJ (eds) *Primer on the automatic nervous system*. Academic Press, San Diego, pp 242-249 (1996).
4. Brunt PW, Mckusick VA FD report of genetic & clinical studies, with a review of the literature. *Medicine*; 49: 343-374 (1970).
5. Maayan C, Kaplan E, Shachar S, Peleg O, Godfrey S. Incidence of Familial Dysautonomia in Israel 1977-1981. *Clin. Genet.*; 32: 106-108 (1987).

6. Slaugenhaupt et al. Tissue Specific expression of a splicing mutation in the IKBKAP gene causes FD. Am. J. Hum. Genet: 68: 598-605 (2001).
7. Anderson et al. FD is caused by mutations of the IKAP gene. Am. J. Hum. Genet. 68: 753-758 (2001).

WARNINGS & PRECAUTIONS

- Reagents supplied in this kit may contain up to 0.1% sodium azide that is toxic if swallowed. Sodium azide has been reported to form explosive lead or copper azides in laboratory plumbing. To prevent the accumulation of these compounds, flush the sink and plumbing with large quantities of water.
- TMB Substrate solution is an irritant of the skin and mucous membranes. Avoid direct contact.
- The Stop Solution contains dilute sulfuric acid (1M), which is an irritant of the eyes and the skin. In case of contact with the eyes, immediately flush them with water. Do not add water to this product. In case of an accident or discomfort consult a physician (if possible, show the bottle label).
- In addition to reagents in this kit, the user may come in contact with other harmful chemicals that are not provided, such as ethidium bromide and EDTA. The appropriate manufacturers' Material Safety Data Sheets (MSDS) should be consulted prior to the use of these compounds.

ASSAY OVERVIEW

The PRONTO[®] procedure detects predefined polymorphisms in DNA sequences, using a single nucleotide primer extension ELISA assay.

- 1 **TARGET DNA AMPLIFICATION:** The DNA fragments that encompass the tested mutations are amplified. This amplified DNA is the substrate for the primer extension reaction.
- 2 **POST-AMPLIFICATION TREATMENT:** The amplified DNA is treated to inactivate free unincorporated nucleotides, so that they will not interfere

with the primer extension reaction.

- 3 **PRIMER EXTENSION REACTION:** In a 96-well thermoplate, each well contains a 5'-labeled primer that hybridizes to the tested DNA next to the suspected mutation site, and a single biotinylated nucleotide species (corresponding to mutant or wild type), which complements the nucleotide base at the tested site. Each post-amplification treated sample is tested in two wells per mutation: the first well of each pair tests for the presence of the mutant allele (*mut*), while the second well tests for the presence of the normal allele (*wt*). The biotinylated nucleotide will be incorporated to the primer in the course of the reaction or not added, depending on the tested individual's genotype.
- 4 **DETECTION BY ELISA:** The detection of the biotinylated primers is carried out by an ELISA procedure: The biotin-labeled primers bind to a Streptavidin-coated ELISA plate and are detected by a peroxidase-labeled antibody (HRP) directed to the 5' antigenic moiety of the primer. A peroxidase reaction occurs in the presence of TMB-Substrate.
- 5 **INTERPRETATION OF THE RESULTS:** The results are determined either visually (substrate remains clear or turns blue) or colorimetrically (substrate remains clear or turns yellow) following the addition of the stop solution.

DISCLAIMER

- Results obtained using this kit should be confirmed by an alternative method.
- Confirmed results should be used and interpreted only in the context of the overall clinical picture. The manufacturer is not responsible for any clinical decisions that are taken.

The user of this kit should emphasize these points when reporting results to the diagnosing clinician or the genetic counselor.

CONTENTS OF THE KIT

FD Amplification Mix.....	1 vial (clear cap)	(1.2 ml)
PRONTO® Buffer 2.....	2 bottles	(3 mL)
Solution C	2 vials (yellow cap)	(130 µL)
Solution D.....	2 vials (red cap)	(100 µL)
ColoRed™ Oil	1 dropper bottle	(13 mL)
Assay Solution.....	1 bottle (green solution)	(100 mL)
Wash Solution (conc. 20x).....	1 bottle	(100 mL)
HRP Conjugate	1 vial	(450 µL)
TMB Substrate	1 bottle	(40 mL)
Stop Solution (1M H ₂ SO ₄).....	1 bottle	(30 mL)
PRONTO® FD Plates.....	2 individually pouched plates	
Detection Plates	2 Streptavidin-coated ELISA plates	

STORAGE AND STABILITY

- Store at 2-8°C. **Do not freeze.**
- Do not use the kit beyond its expiration date (marked on box label). Stability is maintained even when components are re-opened several times.
- Minimize the time reagents spend at room temperature.
- This kit has been calibrated and tested as a unit; **do not mix reagents from kits with different lot numbers.**

ADDITIONAL MATERIALS REQUIRED

- Deionized water (about two liters per kit)
- Thermowell plate or tubes (thin wall) for the post-amplification treatment
- Sterile pipette tips
- Troughs/reagent reservoirs - for use with the detection reagents
- Thermocycler for a 96-well microplate
- Multichannel pipettes (5-50 µL and 50-200 µL)
- Positive displacement pipettes (1-5 µL, 5-50 µL, 50-200 µL & 200-1,000 µL)

- Filtered tips
- ELISA reader with 450 nm filter (optional - 620 nm filter)
- Polaroid camera and color film to record results (optional)
- Automated microtiter plate washer or squirt bottle
- Vortex mixer
- Timer

ASSAY PROCEDURE

1 DNA AMPLIFICATION

- Dispense** 2 µL template DNA (from an initial concentration of about 150 ng/µL) to a thermoplate well or tube.
- Prepare** a Master Mix in a sterile vial, according to the volumes indicated in the table below, plus one spare reaction volume. Add the Taq DNA polymerase to the Master Mix shortly before dispensing the Mix. Gently mix by pipetting in and out several times.

PCR Master mix

Solution	Volume for one sample
Amplification FD™	13.0 µL
Taq DNA Polymerase (5 u/µL)	0.5 µL

The following Taq DNA polymerases (lacking 3' →5' exonuclease activity) were validated for use with this procedure:

- | | |
|----------------|------------------|
| • PHARMACIA | Cat. # 27-0799 |
| • SIGMA | Cat. # D-1806 |
| • ROCHE | Cat. # 1-146-165 |
| • PROMEGA | Cat. # M-1661 |
| • BIOLINE | Cat. # M95801B |
| • PERKIN ELMER | Cat. # M801-0060 |

3. **Dispense** 13.5 µL Master Mix to each sample.
4. **Add** one drop of ColoRed™-Oil to each tube. Do not touch the wells with the tip of the oil bottle. Even when using a thermocycler with a hot lid, it is essential to use oil.
5. **Place** the thermoplate well or tube in a thermocycler previously programmed with the following protocol:

Cycling protocol

1.	94°C	5 minutes	} 35 cycles
2.	94°C	30 seconds	
3.	60°C	30 seconds	
4.	72°C	30 seconds	
5.	72°C	5 minutes	

6. To verify amplification, **subject** 5 µL of the amplified product to electrophoresis in a 2% agarose gel.

Sizes of amplified fragments:

Gene	Mutation	Fragment size
ASPA	693 C>A	162 bp
FACC	IVS4+4A>T	253 bp
ASPA	854 A>C	285 bp
BLM	6-bp del/7-bp ins	660 bp
IKBKAP	2507+6 T>C	802 bp
IKBKAP	R696P	802 bp

2 POST- AMPLIFICATION TREATMENT

! Only 5 µL of each amplified DNA sample will be used to carry out this assay

1. **Prepare** a post-amplification treatment mix shortly before use. Combine in a single test tube the volumes appearing in the following table, multiplied by the number of tested samples, plus one spare volume.

Post-Amplification Treatment

Solution	Volume for one sample
PRONTO® Buffer 2	45.0 µL
Solution C	2.0 µL
Solution D	1.5 µL

3. **Mix** gently by pipetting this solution in and out five times. Do not vortex.
4. **Add** 48 µL of the post-amplification mix into each well or tube containing **5 µL** of each amplified DNA sample.
Ensure that the solution you add becomes well mixed with the DNA sample by pipetting.
5. **Add** one drop of ColoRed™ oil to each tube. Do not touch the tube with the tip of the oil bottle. Even when using a thermocycler with a hot lid, it is essential to use oil.
6. **Incubate** for 30 minutes at 37°C, then for 10 minutes at 95°C in a thermocycler.

If not used immediately, the treated sample can be kept at 2-8°C for a maximum of four hours.

2 PRIMER EXTENSION REACTION

1 Program the Thermocycler as follows:


Cycle	Temperature	Time
Start:	94°C	15 sec.
20 cycles:	94°C	30 sec.
	57°C	10 sec.
End:	18-25°C - Cool down to room temperature	

2 Take a PRONTO® Plate out of its pouch. Notice the color at the bottom of the wells. For each mutation tested, use a pink well (*mut*) and a blue well (*wt*). Mark the plate with the ID numbers of your test.

If you intend to use less than a full plate, you can cut the plate and return the unused portion to the pouch. If you do this, seal the pouch immediately with its desiccant card inside.

3 Dispense 8 µL of post-amplification treated DNA into the first two wells in row A (see Fig. 1). Continue with the remaining samples. It is possible to transfer up to eight samples simultaneously using a multichannel pipette. Ensure that the solution is at the bottom of each well by inspecting the plate from below. Be sure that the well does not contain air bubbles.

Figure 1: Scheme for dispensing post-amplification treated DNA samples into the PRONTO® FD Plate

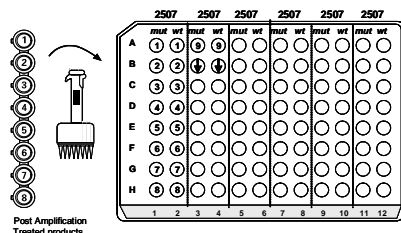

Recommendation:

Use a new set of tips for each column. Alternatively use the same set of tips, but do not touch the bottom of the wells.

- 4 Tilt the plate and add one drop of ColoRed™ Oil to each well. Do not touch the well with the tip of the oil bottle. Even when using a thermocycler with a hot lid, it is essential to use oil.
- 5 Turn on the thermocycler and start the cycling protocol. Insert the plate when the temperature has reached 90°C.
- 6 When the thermal cycling is complete, you can proceed to the ELISA assay, or store the reaction products in the refrigerator and carry out the visualization steps within 24 hours.

3 ELISA ASSAY - COLOR DEVELOPMENT

The ELISA assay consists of the following steps:

- **Binding** the biotinylated primer to the Streptavidin-coated plate.
- **Washing** away the unbound primer.
- **Incubating** with the HRP conjugate.
- **Washing** away the unbound conjugate.
- **Incubating** with the TMB Substrate (color development).

The results of this assay can be determined in one of two ways:

- Visually:** by monitoring the development of the blue color.
- or
- Colorimetrically:** by adding Stop Solution and measuring the absorbance using an ELISA reader at a wavelength of 450 nm (yellow color).

! Before proceeding with the ELISA assay make your choice of visual or colorimetric determination of results.

PREPARATION


- All components used in the detection step should reach room temperature before starting the assay.
- Dilute the 20x Wash Solution to 1x with deionized water
Dilute solution may be kept at 18-25°C for up to one month.
- Peel off the plastic cover of the Detection Plate. Mark the side of the plate with the kit name and test number.

- Place the PRONTO® plate and the Detection plate side by side, oriented in the same direction (see Fig. 2).

TRANSFER TO THE DETECTION PLATE

- Fill a reagent reservoir /trough with the green colored Assay Solution. About 11 mL will be required for a 96-well plate.
- Using a multichannel pipette, **add** 100 µL of Assay Solution to the bottom of each well in column 1 of the PRONTO® Plate. Gently mix by pipetting in and out 3-4 times.
- Without changing tips, transfer** 100 µL from each well in this column to the first column in the Detection Plate (see Fig. 2).
Ensure that the solution at the bottom of all wells of the PRONTO® plate has turned green by inspecting them from below.

Figure 2: Transferring the primer extension products from the PRONTO® Plate to Detection Plate.

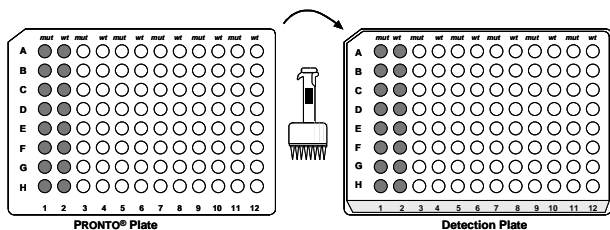

- Repeat** this procedure, using a new set of tips for each column. It is essential to maintain the order of the samples.
10 µL of oil carried over or 10 µL of the sample left behind will not significantly affect the detection process.
- Incubate** for 10 minutes at room temperature (18-25°C).

DETECTION BY ELISA

Procedure	Visual Detection (Blue color)	Colorimetric Detection (yellow color)
6 While the incubation of assay solution is taking place, dilute the Conjugated HRP in Assay Solution. For every detection plate used (96-well), about 11 mL of diluted conjugate is required. This solution should be freshly prepared each time the test is run.	Dilution: 1:100 (110 µL of conjugated HRP into 11 mL Assay Solution)	Dilution: 1:250 (44 µL of conjugated HRP into 11 mL Assay Solution)
7 Empty the plate and wash four times with 350 µL 1x Wash Solution. Ensure that the plate is relatively dry after the last wash step.	√	√
8 Add 100 µL of freshly diluted conjugated HRP to all the wells, with a multichannel pipette.	√	√
9 Incubate at room temperature.	10 minutes	10 minutes
10 Wash the plate as in step 7.	√	√
11 Add 100 µL TMB-Substrate to each well with a multichannel pipette and incubate at room temperature (18-25°C) until blue color appears.	15 minutes	15 minutes
12 Add 100 µL of Stop Solution to each well with a multichannel pipette. The solution will turn yellow immediately.	—	100 µL
13 The results can be documented using a Polaroid camera with color film (for example - Fuji FP-100C), or by reading the absorbance using an ELISA reader (signal wavelength setting).	O.D. 620 nm	—
14 Within two hours read the absorbance using an ELISA reader (single wavelength setting).	—	O.D. 450 nm

VALIDATION OF THE RESULTS


For Visual Detection:

For every mutation site tested, at least one of the wells should develop a deep blue color. Otherwise, the results are invalid for the relevant mutation (Fig. 3).

For Colorimetric Detection:

For every mutation site tested, at least one of the two wells should yield an O.D. ≥ 0.50 reading.

Figure 3: Visual Interpretation of Genotypes

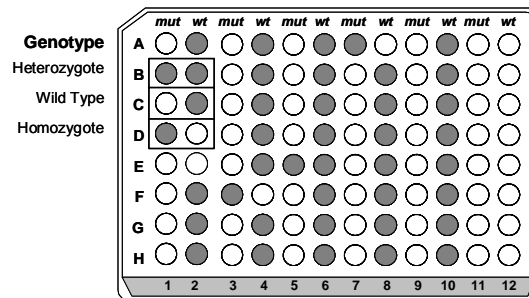


INTERPRETATION OF RESULTS

Important: Heterozygous or homozygous mutant results should be confirmed by retesting. It is recommended to repeat the test with newly extracted DNA.

Criteria for Visual Interpretation

A deep blue color indicates a positive signal, while negative signals appear as a clear or pale blue-colored well (see Fig. 4).

Figure 4: Genotype assignment according to visual inspection of test results

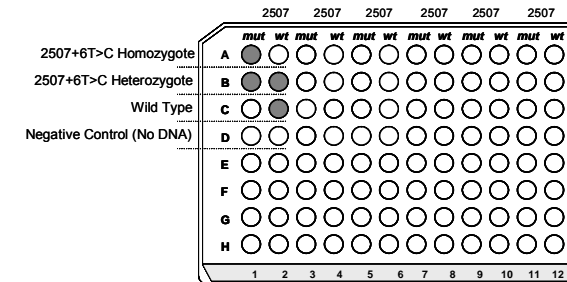

CRITERIA FOR COLORIMETRIC INTERPRETATION

The genotype of each sample is determined according to two criteria:

1. The O.D. values of the *mut* and *wt* wells.
2. The ratio of *mut* / *wt* O.D. values.

Calculate the *mut* / *wt* ratios by dividing the signal of the *mut* well by the signal of the *wt* well.

Identify the correct genotype using the table below:

Genotype	<i>mut</i> well (O.D. 450)	<i>wt</i> well (O.D. 450)	<i>mut</i> / <i>wt</i> ratio
Normal	O.D. ≤ 0.35	O.D. ≥ 0.5	ratio > 0.5
Heterozygote	O.D. ≥ 0.5	O.D. ≥ 0.5	$0.5 < \text{ratio} < 2.0$
Homozygote	O.D. ≥ 0.5	O.D. ≤ 0.35	ratio > 2.0

! Samples with values not included in the above table are considered indeterminate and should be retested.

PRONTO® FD - PROCEDURE SUMMARY

DNA EXTRACTION: from human whole blood, using a validated method.

DNA AMPLIFICATION:

Volumes per reaction: 2 µL Template DNA + 13 µL Amplification Mix + 0.5 µL Taq Polymerase.

Cycling protocol:

94°C 5 min → 35 cycles of {94°C 30 sec. / 60°C 30 sec. / 72°C 30 sec.} → 72°C 5 min.

POST-AMPLIFICATION PROCEDURE:

- Volumes per reaction:

PRONTO® Buffer 2	45.0 µL
Solution C	2.0 µL
Solution D	1.5 µL
- Pipette in and out to mix.
- **Add** 48 µL into each well containing 5 µL amplified product, mix well.
- **Add** one drop of ColoRed™ oil.
- **Incubate** 30 minutes at 37°C, then 10 minutes at 95°C.

PRIMER EXTENSION REACTION:

- **Dispense** 8 µL of each post-amplification treated DNA into two wells of the PRONTO® Plate.
- **Add** one drop of ColoRed™ oil.
- **Start** the cycling protocol:
- 94°C 15 sec → 20 cycles of {94°C 30 sec. / 57°C 10 sec.} → Cool.
- **Insert** the PRONTO® Plate in the thermocycler when the temperature is 90°C

DETECTION:

- **Add** 100 µL Assay Solution to each well in the PRONTO® Plate and mix.
- **Transfer** 100 µL from each well of the PRONTO® Plate to the identical position in the Detection Plate. Incubate 10 minutes at RT.
- **Empty** the wells and wash four times with 350 µL of 1x Wash Solution.

	Visual Detection	Colorimetric Detection
Add 100 µL of Conjugated HRP to every well and incubate for 10 minutes at RT.	Dilution 1:100	Dilution 1:250
Empty the wells and wash four times with 350 µL of 1x Wash Solution.	√	√
Add 100 µL of TMB Substrate to each well and incubate at RT for:	15 minutes	15 minutes
Add Stop Solution	—	100 µL per well
Read O.D. at:	620 nm	450 nm

For troubleshooting guide, please refer to our website:

www.prontodiagnostics.com/ts

PRONTO is a registered trademark

COLORED is a trademark of Pronto Diagnostics Ltd.

Copyright © Pronto Diagnostics Ltd.

The PRONTO® Technology is covered by US patent 5,710,028, by European patent 0648222 and by corresponding national patents.

Produced by Pronto Diagnostics Ltd.

Kiryat Atidim

Building 3, 3rd floor

POB 58129

Tel Aviv 6158002

Israel

Tel: +972.73.2126155

Fax: +972.73.2126144

Customer Service: E-Mail: info@prontodiagnostics.com

Authorized EU Representative:

MedNet GmbH,

Borkstrasse 10, 48163 Münster,

Germany

Tel: +49.251.32266-0

Fax: +49.251.32266-22

MC9948 05.EN.02